

5. FOLDING INSTRUCTIONS

1. Lower mast to lowest position and press the RED safety stop button on the LINAK Jumbo Care control unit.



2. Close legs to narrowest position.




3. Secure the spreader bar to the mast with the wire harness located under the lower part of the main actuator.



4. Pull black lever on base between legs to close completely.




5. Undo base fastening lever bolt and remove bolt.

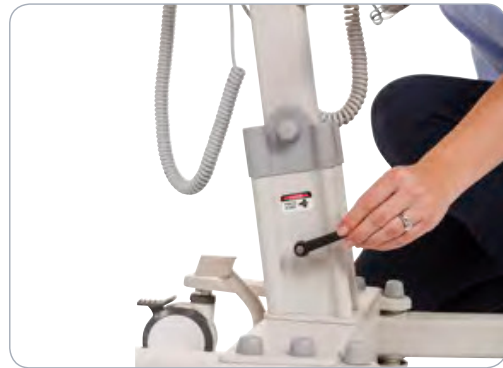

6. Fold mast down carefully and secure to legs with cords located under the end of legs.



7. Re-insert base fastening lever bolt & secure to avoid misplacing.

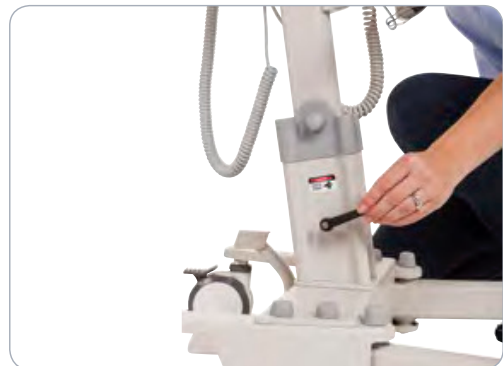

8. Lifter can be moved by the handles, using the rear castor wheels to wheel the lifter. Always use safe lifting techniques.

