

**WendyLett**  
SystemRoMedic™




WendyLett Sliding Sheets  
distributed in New Zealand by:




# Sheet guide

Choose the correct sliding zone on the sheet depending on user function

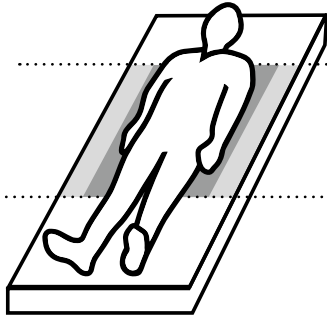


### Sliding zone Hip

For more active/lighter users who only need the sliding function at hip level.

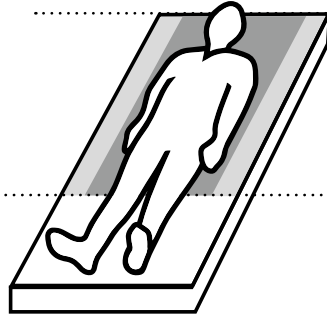

### Sliding zone Head-Hip


For less active/heavier users who need the sliding function from hip level and up.

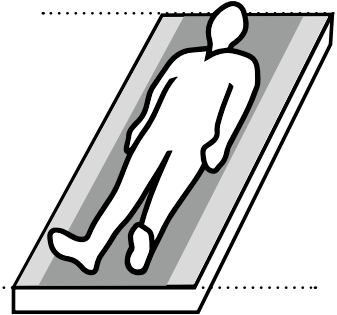

### Sliding zone Head-Feet

For users who are passive/in need of more intensive care where the sliding function is required under the entire body.

Choose the correct sliding sheet based on bed width

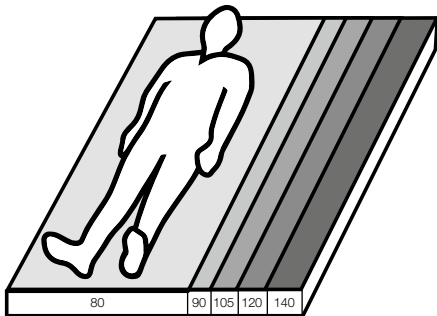

80cm 31.5"	90cm 35.4"	105cm 41.3"	120cm 47.2"	140cm 55.1"
---------------	---------------	----------------	----------------	----------------

Bed width

The mattress dimensions are important for optimising the sliding surface of the sheet and stimulating the activity of the user. For our draw sheets and sliding draw sheets, there is no recommended mattress width.

Find the correct function



### Incontinence protection

Incontinence protection is a washable incontinence sheet placed on top of an ordinary sheet and/or other bedding. Incontinence protection is available with or without a sliding surface on the bottom. The material is highly absorbent and distributes liquid over a larger surface.



### Handle

Some sliding draw sheets are also equipped with handles that can be connected to lifts. This facilitates easier positioning, takes care of lower body hygiene and facilitates harness attachment.



### Elastic


Sliding sheets with this symbol have elastic that is pulled over the mattress. Other sheets have side sections that are tucked in under the mattress.

Care instructions.


Choose the right product bearing in mind the care instructions.




# Select the correct combinations

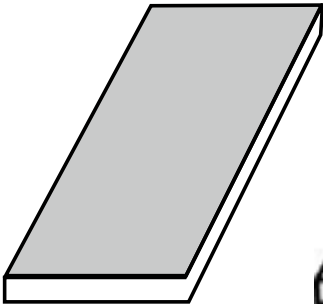


**Mattress cover**  
with barrier layer. Placed on the top of the mattress.

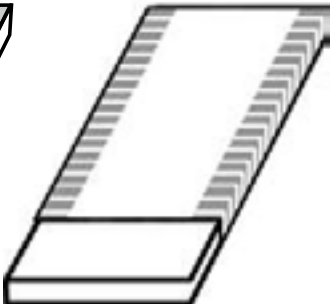

**Sliding sheet**  
Satin for the centre section and non-slip material for the side sections. Slide in the direction of the pattern.




**Sliding sheet + draw sheet**  
Facilitates turning and reduces the heat from the satin of the sliding sheet. Slide in the direction of the pattern.

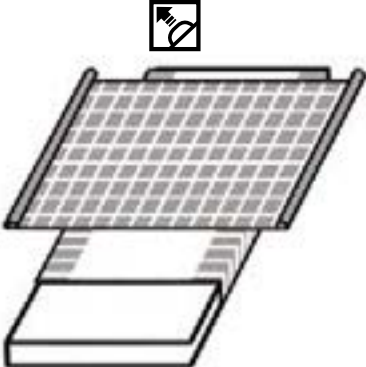


**Sliding sheet + sliding draw sheet**  
Both the sheets have sliding surfaces on both the bottom and the top. Slide in the direction of the pattern.


# How to use the combinations



For patients able to turn over on their own, often a sliding sheet that improves the slide in the zones where this is needed is sufficient.



A combination of a sliding sheet and a draw sheet is used when the user is less active, heavier/requires more intensive care.

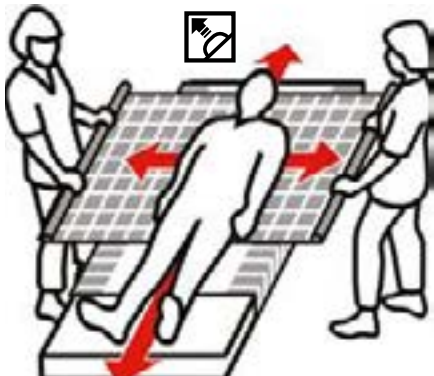

The combination of a sliding sheet and a sliding draw sheet is used when the user is passive, heavy/in need of more intensive care. This combination is often used with a lift, during hygiene work and when attaching a harness.

# Find the right sheet

Example from item list:

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
1661	WendyLett	120/47.2	120/47.2	120/47.2	95/37.4	Grey/white striped	Sliding sheet	Possible info
Product name	Item number	The sheet is suitable for this bed width	Dimensions (length + width)	Zone/sliding surface	Product colour	Product function	Other info	Repetition of item number Special properties Sheet + sliding zone on the mattress Art. No. 1661

## POSITIONING

### Sliding aids

#### WendyLett portfolio



WendyLett  
Sliding Sheet  
page 19



WendyLett2Way  
Draw Sheet  
page 20



WendyLett4Way  
Sliding Draw Sheet  
page 21

+



WendyLett inco range  
WendyLett 4Way, inco  
Sliding Draw Sheet  
page 21



WendyLean  
pillow case  
page 23


#### EasyRoll portfolio



EasyRoll  
Sliding Sheet  
page 23



EasyRoll 4way  
Sliding Drawsheet  
page 24

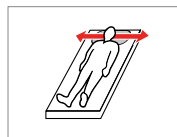

EasyRoll Pillow Case  
page 25



## Sliding aids

Our WendyLett products are our patented high-quality systems designed to facilitate repositioning higher up in bed, turning and minor repositioning in bed. Together, Direct Healthcare Group's three WendyLett sheets constitute a well thought out system of sliding and draw sheets that complement each other and contribute to making day-to-day life easier for both users, healthcare professionals and private carers.

### WendyLett Sliding Sheet

WendyLett is a satin sliding sheet. Sliding takes place sideways, in the direction of the stripes. WendyLett is for users who have trouble turning over in bed. WendyLett has a smooth centre section, which means very little energy is required to turn over. It is very easy for the user to slide sideways, but they are stopped by a non-slip material on the side sections so that they do not slide out of the bed. The material also prevents the user from sliding down the bed.

When not in use, the WendyLett can be tucked in around the bed like a normal sheet, with the smooth surface in the centre of the bed. For users who need more help, WendyLett can be combined with the WendyLett2Way draw sheet or the WendyLett4Way sliding draw sheet. WendyLett is available in several different versions and sizes.



### Product overview WendyLett Sliding Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Sheet	Sheet length	Sliding zone
1609	WendyLett	80–90/31.5–35.4	100/39.4	200/78.7	60/23.6	White	Sliding sheet		Art. No. 1609		
1614	WendyLett	80–90/31.5–35.4	140/55.1	200/78.7	60/23.6	White	Sliding sheet		Art. No. 1614		
1629	WendyLett	80–90/31.5–35.4	100/39.4	200/78.7	60/23.6	Grey/white striped	Sliding sheet		Art. No. 1629		
1634	WendyLett	80–90/31.5–35.4	140/55.1	200/78.7	60/23.6	Grey/white striped	Sliding sheet		Art. No. 1634		
1641	WendyLett	90/36	200/78.7	90/36	70/27.6	Grey/white striped	Sliding sheet		Art. No. 1641		
1639	WendyLett	105/41.3	200/78.7	200/78.7	70/27.6	Grey/white striped	Sliding sheet		Art. No. 1639		
1651	WendyLett	105/41.3	200/78.7	105/41.3	80/31.5	Grey/white striped	Sliding sheet		Art. No. 1651		
1667	WendyLett	120/47.2	100/39.4	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1667		
1668	WendyLett	120/47.2	140/55.1	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1668		
1669	WendyLett	120/47.2	200/78.7	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1669		
1661	WendyLett	120/47.2	200/78.7	120/47.2	95/37.4	Grey/white striped	Sliding sheet		Art. No. 1661		
1671	WendyLett	140/55.1	200/78.7	140/55.1	110/43.3	Grey/white striped	Sliding sheet		Art. No. 1671		
1681	WendyLett	–	200/78.7	365/143.7	70/27.6	Grey/white striped	Sliding sheet	Extra wide to allow for optional placing of sliding surface	Art. No. 1681		

**Material:** Polyester, cotton, satin

**Care instructions:** Do not use fabric softener.

Avoid tumble drying to ensure optimum service life of the material.



CE Class 1 medical device



POSITIONING

Sliding aids

WendyLett2Way Draw Sheet


WendyLett2Way is a draw sheet made from a material adapted for WendyLett sliding sheets. Sliding takes place sideways, in the direction of the stripes, which means that the user will not slide down in the bed. WendyLett2Way is placed on top of the WendyLett sliding sheet. The combination makes it easy for the care assistant to turn the user sideways. WendyLett2Way is available in several versions and sizes.



Product overview WendyLett2Way Draw Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1620	WendyLett2Way	–	90/35.4	180/70.9	–	White	Draw sheet	Art. No. 1620	
1631	WendyLett2Way	–	100/39.4	200/78.7	–	Grey/white striped	Draw sheet	Art. No. 1631	
1636	WendyLett2Way	–	140/55.1	200/78.7	–	Grey/white striped	Draw sheet	Art. No. 1636	

**Material:** Polyester, cotton, satin  
**Care instructions:** Do not use fabric softener.  
Avoid tumble drying to ensure optimum service life of the material.

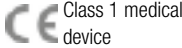

Our WendyLett products

- Thin, high-quality fabric that breathes
- Suitable for users at risk of developing pressure ulcers
- Clear pattern shows slide direction





## Sliding aids

## WendyLett4Way Sliding Draw Sheet

WendyLett4Way is a sliding draw sheet that can be combined with a WendyLett sliding sheet to facilitate sliding both sideways and up/down in the bed, as the grid pattern indicates. WendyLett4Way is placed on top of the WendyLett sliding sheet. The combination allows positioning in four directions and thereby facilitates turning in bed, repositioning higher up in bed and minor repositioning while lying in bed, even with large, heavy or completely passive users. WendyLett4Way is available in many versions and sizes.



## Product overview WendyLett4Way Sliding Draw Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1657	WendyLett4Way	–	140/55.1	190/74.8	–	Grey/white checked	Sliding draw sheet	2 x 7 handles	Art. No. 1657
1637	WendyLett4Way	–	140/55.1	200/78.7	–	Grey/white checked	Sliding draw sheet		Art. No. 1637
1647	WendyLett4Way	–	200/78.7	200/78.7	–	Grey/white checked	Sliding draw sheet		Art. No. 1647
1652	WendyLett4Way	–	140/55.1	260/102.4	–	Grey/white checked	Sliding draw sheet	Wider mattress	Art. No. 1652

**Material:** Polyester, cotton, satin

**Care instructions:** Do not use fabric softener.

Avoid tumble drying to ensure optimum service life of the material.



Class 1 medical device



## Product overview WendyLett4Way Inco Sliding Draw Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1648	WendyLett4Way, inco	–	75/29.5	200/78.7	–	Grey/white checked	Sliding draw sheet	Extra fabric on both sides	Art. No. 1648
1649	WendyLett4Way, inco	–	140/55.1	120/47.2	–	Grey	Sliding draw sheet	2 x 7 handles	Art. No. 1649

**Material:** Polyester, cotton, satin

**Care instructions:** Do not use fabric softener.

Avoid tumble drying to ensure optimum service life of the material.



Class 1 medical device





0800 658 814 | [uslequipment.co.nz](http://uslequipment.co.nz)  
[uscustomerservices@uslmedical.co.nz](mailto:uscustomerservices@uslmedical.co.nz)

