


NJ PHILLIPS PRECISION INSTRUMENT

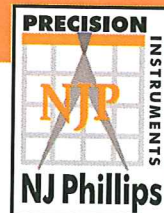

# Ear Syringe


## LEVER ACTION

For use in the removal of cerumen along the outer ear canal.


### CAUTION:

1. DO NOT insert the ear tip end into the outer ear canal.
2. Take due care - rapid jets of water directed at the tympanic membrane could be harmful.
3. This device should only be used by a health care professional familiar with ear syringing techniques.

TGA APPROVED #104681

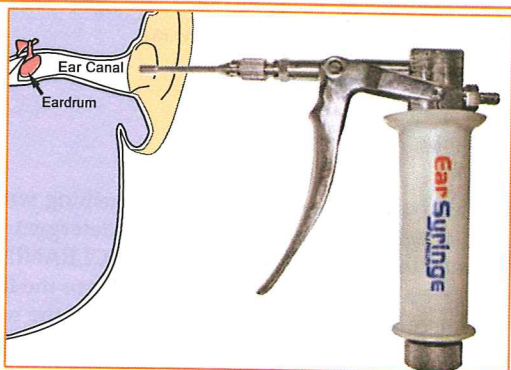

### ASSEMBLING EAR TIP UNIT:

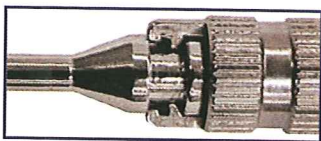

1. Loosen needle nut until reasonably loose.
2. Slide needle tip base into the lower slot of the needle nut.
3. Screw the needle nut tightly back onto the main threaded mount.

### CLEANING OR STERILISATION:

The following items may be autoclaved:

Ear tip needle; needle nut; the entire lever action hand piece - however it is important to apply a small drop of castor oil to any moving part post autoclave as good maintenance. To apply castor oil to moving parts within the handle grip, carefully unscrew counter clock wise.

*It is IMPORTANT to make sure the thread is properly engaged when reconnecting the handle grip. To reconnect handle grip gently pull lever trigger until the handle grip cover slides up and over the handle body and screw clock wise until tight - DO NOT OVER TIGHTEN!*


*It is unnecessary, nor desirable, to dismantle any part of the lever action unit for any purpose other than to lubricate moving parts.*

**Note:** DO NOT autoclave the hose piece, rather wash in a good quality Medical Equipment & Instrument detergent, then hang to drip dry.

## LIQUID MEASURE DISPERSMENT SETTINGS:



CIRCULAR NUT

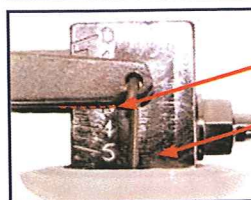

DOSE  
SETTING

BODY POST

1. To set liquid dispersment adjust the circular nut located at the bottom of the handle. Turn the circular nut clockwise or counter clockwise to adjust the (ml) calibrations printed at the top of the body post. Adults generally require 3-5 (ml) setting. The dose is set from the bottom of the lever arm.
2. Prime instrument to ensure all air is removed prior to use on the patient.

## CARE AND MAINTENANCE

1. Clean ear tip and needle nut by brushing with a non wire brush in a good quality Medical Equipment & Instrument detergent prior to autoclave process.
2. Autoclave the unit as indicated in 'CLEANING OR STERILISATION' when required.
3. Draw a small quantity of castor oil into the lever action unit through the inlet fitting (where the hose attaches to the back of the unit) to maintain a well-lubricated action.
4. Place a few drops of castor oil in all moving parts to ensure a smooth action.

### NJ PHILLIPS EAR SYRINGE - NATIONAL DISTRIBUTION NETWORK

#### NEW SOUTH WALES / AUSTRALIAN CAPITAL TERRITORY

D.V.M. Medical Supplies  
7 Doyle Avenue  
Unanderra N.S.W 2526  
Contact: Mr. Bob Glover  
Tel: 1800 10 10 33  
Mob: 0412 095 186

Medical Access Pty. Ltd.  
226 Mc Cann Road  
Rossmore N.S.W 2171  
Contact: Ms. Deborah  
& Grant Cotter  
Tel: (02) 960 66569  
Fax: (02) 960 66036  
Mob: 0417 671 627

#### VICTORIA

Warner and Webster  
Pty. Ltd.  
273 Inkerman Street  
St Kilda VIC 3182  
Contact: Customer Service  
Tel: (03) 9534 8039  
Fax: (03) 9534 4079

QUEENSLAND  
Medical And Surgical  
Requisites Pty. Ltd.  
33 Fulcrum Street  
Richlands, Brisbane QLD 4077  
Contact: MEDI + Plus -  
Service Advisor  
Tel: (07) 3217 0966  
Fax: (07) 3375 1744


SOUTH AUSTRALIA /  
NORTHERN TERRITORY  
Mc Neil Surgical Pty Ltd.  
29-31 Magill Road  
Stepney SA 5069  
Contact: Mr. Kris Thrower  
Tel: (08) 8363 0888  
Fax: (08) 8363 2137

WESTERN AUSTRALIA  
AMA Medical Products  
14 Stirling Highway  
Nedlands WA. 6909  
Contact: Customer Service  
Tel: (08) 9273 3022  
Fax: (08) 9273 3017

#### TASMANIA

Tasmanian Medical  
Supplies  
40 Brisbane Street  
Hobart TAS 7000  
Contact: Mr. Ben Jones  
Tel: (03) 623 49777  
Fax: (03) 623 48452

As our policy is one of continuous improvement, the manufacturer reserves the right to alter these specifications at any time.  
All products produced by NJ Phillips Pty Limited, are identified by a unique batch number. This identification number is affixed to the product to allow traceability by the manufacturer and must not be removed if integrity is to be maintained.

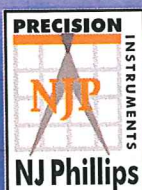

MADE IN AUSTRALIA

NJ PHILLIPS Pty Limited ABN 36 000 082 002

POSTAL ADDRESS: LOCKED BAG 8, GOSFORD NSW 2250 AUSTRALIA

TELEPHONE: (02) 4340 2044 FAX: (02) 4340 1991 EMAIL: [njp1@njphillips.com.au](mailto:njp1@njphillips.com.au)  
[www.njphillips.com](http://www.njphillips.com)